


Aluminum Mouldable-Splint

DW-FA012



Usage for the flexible rolled splint

If prolonged use is anticipated (more than a few hours), place absorbent material, such as cotton cloth, between the flexible splint and the skin to prevent skin irritation and odor. Also, to prevent uncomfortable pressure points during prolonged use, place soft padding (such as gauze pads) around all bony prominences. When cutting the flexible splint avoid using serrated scissors as they may produce sharp edges. After cutting the flexible splint, roll the cut end over or apply tape to cover any exposed metal.

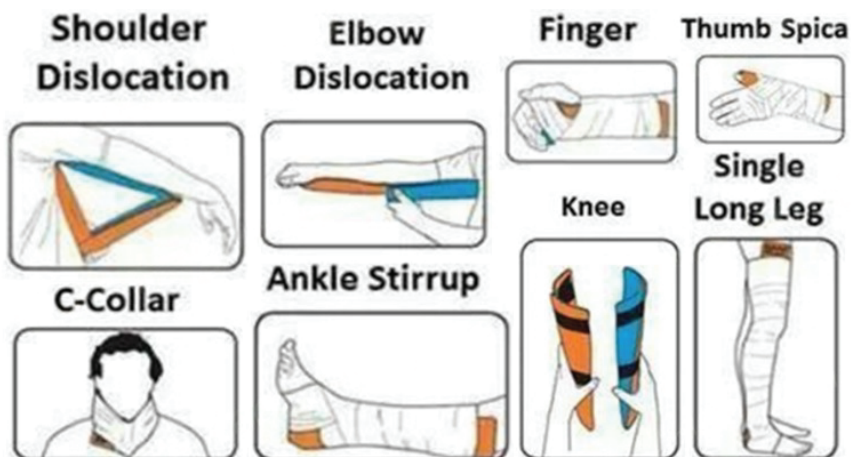

MODEL	EXPANDED SIZE	PACKING SIZE	QUANTITY	G.W
DW-FA012	11 x 45cm	62 x 33 x 24cm	100	7.5kg
	11 x 61cm	57 x 44 x 24cm	120	12kg
	11 x 91cm	43 x 43 x 45cm	100	15.5kg